


Mai-Cruise™

User Manual

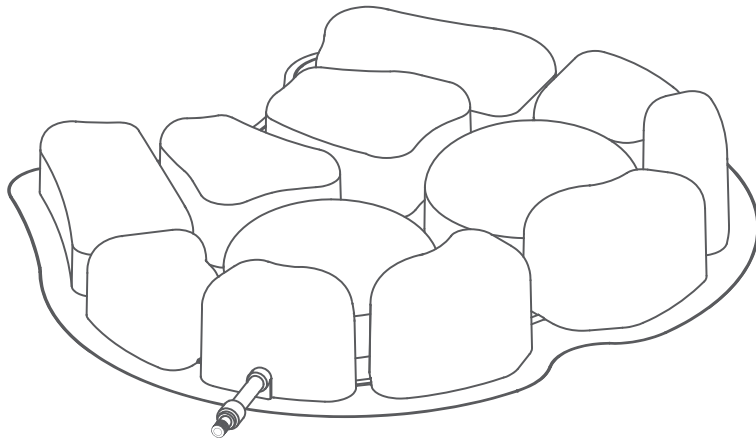

SINGA

Ideal for Life Science

ISO 9001:2008 、 ISO13485:2003

Mai-Cruise™
Floatation Cushion
1-Year Limited Warranty

SINGA TECHNOLOGY CORPORATION covers any defects in materials or workmanship of the cushion, under normal use and specified conditions, for a period of one year from the date the cushion was originally purchased as a new cushion, no matter who is the present owner.

Within a reasonable amount of time after you return the cushion to us, we will repair any defect in material and workmanship and ship it back to you. We will not repair a cushion free of charge if it has been misused or damaged by an accident.

You must send the cushion to us with the original sales ticket and you must pay all delivery costs. It must be sent to **SINGA TECHNOLOGY CORPORATION**. You must return the warranty registration card, attached to this brochure, or register the warranty online, within thirty days of purchasing the cushion. We will not require the original sales ticket if you or the previous owner, have completed and returned the warranty registration card / registered the warranty registration form online, and we have it on file. Failure to submit the original sales ticket or failure to have the warranty registration card in our file voids this warranty.

SINGA shall not be responsible for incidental or consequential damages with respect to economic loss or injury to person or property, whether as a result of breach of warranty, negligence or otherwise. **SINGA** will not be responsible for the results of careless handling or unreasonable use.

Any impelled warranties arising by operation of law shall be limited in duration to the terms of this warranty. This warranty gives you specific legal rights, and you may have other rights that vary from state to state.

For any further information, please contact our Customer Services Department.



SINGA TECHNOLOGY CORPORATION

No.35, Keji 1st Rd., Guishan Dist., Taoyuan City 333, Taiwan

TEL : +886 3 328 8328 FAX : +886 3 327 1400

<http://www.singa.com.tw> e-mail:service@singa.com.tw




PACKAGING


BEFORE USE THE CUSHION, PLEASE CHECK THE CONTENT OF PACKAGE.

- ☺ Mai-Cruise cushion.....x 1
- ☺ Cushion cover.....x 1
- ☺ Elastic Strap.....x 2
- ☺ Repair kit.....x 1
- ☺ User manual.....x 1

INSTALLATION

You can carry out the cushion installation by following the necessary steps carefully as below:

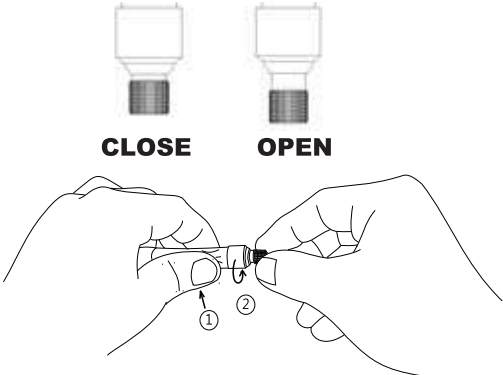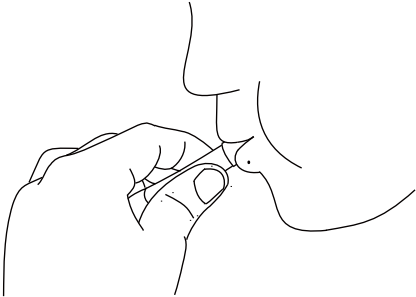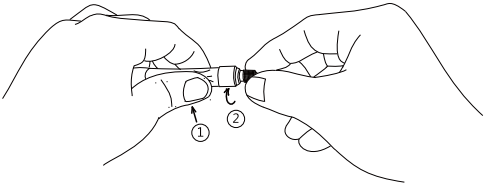	<p>Step 1:</p> <p>Turn cushion valve counterclockwise to open.</p> <p>Counterclockwise : to open the valve</p> <p>Clockwise : to close the valve</p>
	<p>Step 2:</p> <p>Inflate cushion by mouth until the cushion is filled with air.</p>
	<p>Step 3:</p> <p>Turn valve clockwise to close.</p>

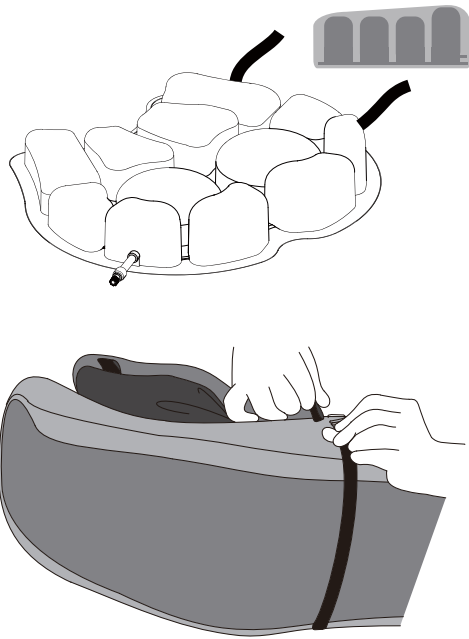


Step 4:

Connect 2 elastic straps with Mai-Cruise™ cushion and assemble Mai-Cruise™ centrally on your motorcycle seat.

Adjust the straps to proper tightness.

ATTENTION:

Make sure the air cells facing upward and the valve is located on the left back side.

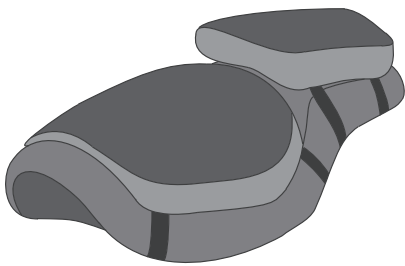


Step 5:

Sit on the seat in your normal riding position and place both feet on pegs.

Open the valve (counterclockwise) and release the air until you sink into the cushion. Then close the valve.

ATTENTION:

The ideal inflation is about 1.5 cm/ 0.6 inch of the cushion. DO NOT use an over inflated cushion.

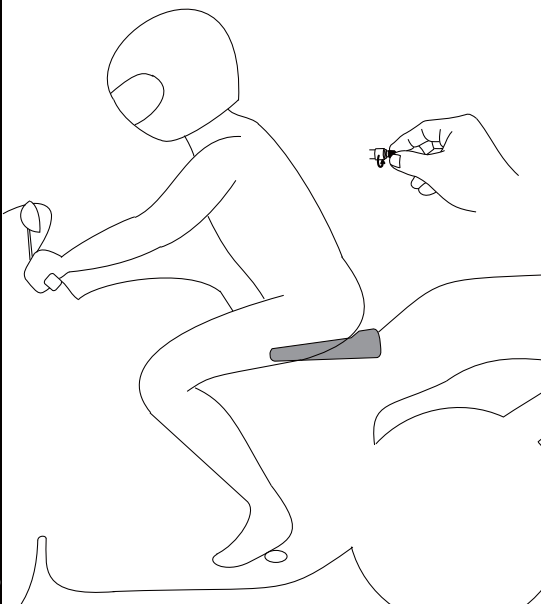

Step 6:

Check the inflation level by moving your body on the cushion back and forth. You should be able to feel the air shift slightly beneath you.

If you bottom out on the base of the seat, open the valve, add more air in and adjust again.

RECOMMENDATION

1. Regulate the cushion internal pressure every 5 days to keep the good condition for seating. To adjust the cushion air pressure, especially while the ambient temperature or altitude changes a lot.
2. Ensure to wear cushion cover onto the cushion before using the Mai-Cruise™ cushion, and wear pants just in case of skin allergy.
3. To avoid any incidents when sitting on Mai-Cruise™ cushion, please do use our slip-resistant cover; please DO NOT sit on the cushion without cover!

CLEANING AND WASHING

CUSHION

To clean the cushion, deflate, close valve and place in a large sink. Using a soft plastic bristle brush or washcloth, gently scrub the air cells and spaces in between. Rinse with clean water, then towel or air dry.

CAUTION

- DO NOT use non-neutral detergent
- DO NOT wash cushion in a washing machine
- DO NOT use a heat or steam autoclave
- Away cushion from direct sunlight

COVER

To clean the cover, machine wash in cold. Rinse with clean water, then air dry.

CAUTION

- Remove straps before putting into washing machine
- DO NOT use drier, heater or bleach
- DO NOT iron

REPAIRING

Any damage of the cushion happen during the warranty, please contact our company for the after service.

WARNING

1. Keep the cushion away from sharp objects and hot ashes.
2. Avoid exposing to fire flame and direct sunlight.
3. The weight limit on the cushion is 200 kgs, DO NOT overload.
4. The cushion should be used individually.

CONSUMING PARTS

Model Name	Condition to change
Cushion Cover	Damage under normal use or Zipper broken
Elastic Strap	Loose, broken, or missing
Valve	When the valve becomes rusty